# You Dont Have To Be A Trained Paramedic To Save A Life



**iPAD** The Intelligent Public Access Defibrillator

A British Heart Foundation nominated device



Issue Three

# iPAD SP1

### **Overview**

The iPAD – SP1 will redefine your expectations for a defibrillator. It has many new innovative features including pre-connected smart electrodes, "i" button technology, ambient noise detection, but most of all, its designed to be used by minimally trained individuals.

#### 1) Patient Mode Switch

- Easily switch from adult to child mode without changing pads
- Safety cover prevents accidental switching.

#### 2) Speaker

- Ambient noise detector measures level of background noise and adjusts the volume of the voice prompts accordingly.
- Ideal for noisy environments such as public places, factories, warehouses, schools etc.

#### 3) Pad Connector

#### 4) Power Button

#### 5) Shock Button

#### 6) Smart Pads Storage

 Electrode pads are stored, pre-connected, in a clear compartment on the underside of the unit.

#### 7) Disposable Battery Pack

- High Capacity
- Type DC 12 Volt 4.2Ah, Lithium Manganese Dioxide Capacity
- Minimum 200 shocks (150J)
- Lifespan 5 years assumes unit is kept within operable temperature range and after the initial check, the unit is left in standby mode.

#### 8) Data Recording

- IIrDA Port Wireless Transmission of data to PC
- SD Card Port Data can be transferred easily
- Internal Memory (Nand-Flash) ECG, Event
- Storage Capacity Multi recording
- 5 events / max 3 hours
- Data review
- program CU-EX1
- Software (for PC)



The iPAD SP1 defibrillator from CU Medical Systems is designed for use by people with minimal training in the event of a cardiac arrest. Easy to use with built in adult/child switch.

#### 9) Visual Indicator

- Visual Indicator shows at a glance
- Battery Life
- Unit Status
- Pad Status

#### 10) Information Button

'i' button performs the following functions;

- Allows responder to select if CPR metronome is heard during CPR
- Checks for faults and errors
- Provides 'handover' information to Ambulance crew (usage time and shocks delivered)
- Allows the SP1 to be programmed with CPR protocols such as compression rate, number of compressions, breaths and cycles etc, ensuring that the SP1 is always up to date
- Downloads data for review
- Checks the units software version

#### 11) Indicator Lights

- Indicator Lightsoperate in time with the voice prompts to prompt the responder with;
   Pad placement
  - Staying clear of casualty
  - CPR compressions





## **AED Comparison Table**

	IPAD SP1	Cardiac Science G3 Pro	Philip/Laerdal FR3 Pro	ZOLL AED Plus	Physio Control Lifepak CR+	Heartsine Sa- maritan
Electrode Tested For Presence	<b>~</b>	<b>~</b>	<b>~</b>	×	×	×
Daily, Weeky, Monthly Self Tests	<ul> <li></li> </ul>	¥	✓	<ul> <li></li> </ul>	<b>~</b>	<ul> <li>✓</li> </ul>
All Tests Automated	<b>v</b>	<ul> <li></li> </ul>	<ul> <li>Image: A start of the start of</li></ul>	<ul> <li></li> </ul>	<b>v</b>	<b>~</b>
Status Indicator Shows Clinical Readlines	~	~	<ul> <li></li> </ul>	<ul> <li></li> </ul>	<b>~</b>	<b>~</b>
Battery Capacity Gauge	~	<ul> <li></li> </ul>	×	×	×	×
Pad Life Indicatior	~	×	×	×	×	×
Used Pad Dectection	~	~	×	×	×	×
Pad Life From Manufacture	30 months	2 Year	2 Years	5 Years	2 Years	3.5 Years
Uses Same Sets of Pads for Child and Adult	<ul> <li></li> </ul>	×	✓	×	×	×
Ambient Noise Detector	<ul> <li></li> </ul>	×	×	×	×	×
Intergarted Pad Storage	<b>v</b>	×	×	<ul> <li></li> </ul>	<b>&gt;</b>	ТВС
CPR Dectection	<b>v</b>	×	×	<ul> <li></li> </ul>	×	×
Illimunated Prompts	<b>v</b>	×	<b>~</b>	<ul> <li></li> </ul>	<b>&gt;</b>	<b>~</b>
Internal Mermory Capacity	15 Hours	1 Hour	TBC	7 Hours	20 Minutes	45 Minutes
Data Transfer Method	IR / SD Card	Direct Connect	Direct Connect	Direct Connect	Direct Connect	Direct Connect
User Reprogrammable	<ul> <li></li> </ul>	¥	✓	<ul> <li></li> </ul>	<b>~</b>	<ul> <li>✓</li> </ul>
CPR Metronome	<ul> <li></li> </ul>	¥	✓	<ul> <li></li> </ul>	<b>~</b>	<ul> <li>✓</li> </ul>
Paramedic Handover Information	<ul> <li></li> </ul>	×	×	×	×	×
Fluid Ingress Protection	IP 55	IP 24	IP 55	IP55	IP 55	IP 56
Battery Capacity (Shocks/Shelf)	200/5 years	290/4 years	/5 years	/5 years	/2 years	/3.5 years
Carry Case Supplied As Standard	¥	×	×	~	<b>~</b>	~
Manual Self Test Option	¥	×	×	×	×	×
All Status Indicators Visable When Case is Closed	<ul> <li></li> </ul>	×	×	×	×	×

## **Ordering Details**

CodeDescription1844iPAD - CU SP1 GENUINE OEM

**Contents** 2 x AED Starter kits 1 x AED Carry Case



# Call us on **01305 769 969**, or visit **www.siriusbusinessservices.co.uk** for more information.



Sirius Business Services Ltd,

Bothwell House, 116 High Street, Wyke Regis, Weymouth, Dorset DT4 9NU Tel: 01305 769969 Reg. No. 4115319 | VAT No. 768 8310 90